
**UK
A**

CE

aidapt[®]

Jar and Bottle Grip Opener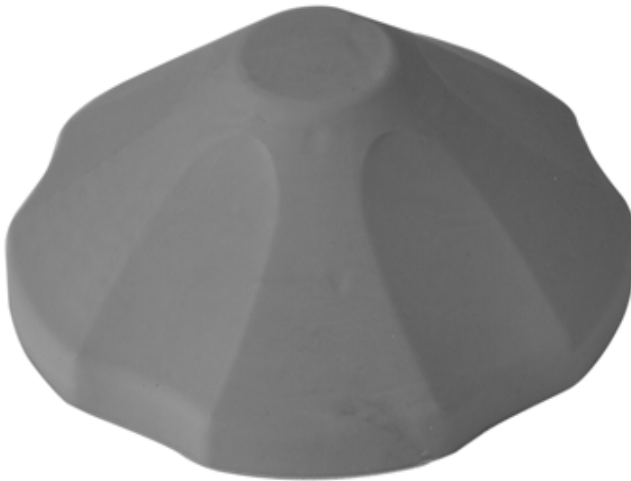

VM992AA

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Jar and Bottle Grip Opener from Aidapt. This product is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The Jar and Bottle Grip Opener is constructed from a non-slip, rubber material. It is very flexible and can be used with many sizes and types of bottle lids and caps as well as cupboard doors and taps.

USING YOUR JAR AND BOTTLE GRIP OPENER

Position the dome-shaped bottle opener over the bottle top or lid and twist to open or tighten.

Please note: product may not be effective when wet. Do not subject to temperatures over 50 degrees Celsius.

CLEANING

When your product gets dirty and dusty, simply restore the Non-Slip properties by washing in warm soapy water and allowing it to dry away from direct heat.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai France (Altai Europe Ltd)