## Universal Jar and Bottle Opener

**VM992AC**

## Usage and Maintenance Instructions

## **INTRODUCTION**

Thank you for deciding to purchase the Jar and Bottle Opener from Aidapt. The Jar and Bottle Opener is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

## **BEFORE USE**

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## **INTENDED USE**

Enables the user to easily open jar lids.

## **INSTRUCTIONS FOR USE**

The Jar and Bottle Opener is simple to use:

- Grip the Jar and Bottle Opener with the ergonomically-designed handle and place the product over the jar lid. Position the Jar and Bottle Opener so that it grips the jar lid and pull/push the handle to twist the jar lid off the jar.

## **CLEANING**

Clean your Jar and Bottle Opener using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

The Jar and Bottle Opener can also be cleaned on the top tray of a domestic dishwasher on a standard/eco or cool program.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

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**Altai France (Altai Europe Ltd)**