Bendable Cutlery

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Bendable Cutlery from Aidapt. The Bendable Cutlery is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

VM914 Bendable Cutlery Set

VM911 Bendable Fork

VM910 Bendable Spoon

VM912 Bendable Rocker Knife

BEFORE USE

Carefully remove all packaging. Avoid using any knives of other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INSTRUCTIONS FOR USE

The Bendable Cutlery is simple to use:

- All items (knife, fork and spoon) feature a bendable handle, which can be bent to an angle that makes it easier and more comfortable for users with limited dexterity.
- The Bendable Fork and Spoon can then be used in the standard way for dining.
- The Bendable Rocker Knife additionally features a rocking/cutting action for single-handed cutting of food.

CLEANING

Clean your Bendable Cutlery using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

The Bendable Cutlery can also be cleaned on the top tray of a domestic dishwasher on a standard/eco or cool program.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

Usage and Maintenance Instructions

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VM914	Lot de Couverts Pliables
VM911	Fourchette Pliable
VM910	Cuillère Pliable
VM912	Couteau Pliable

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**Altai Germany (Altai Europe Ltd.)